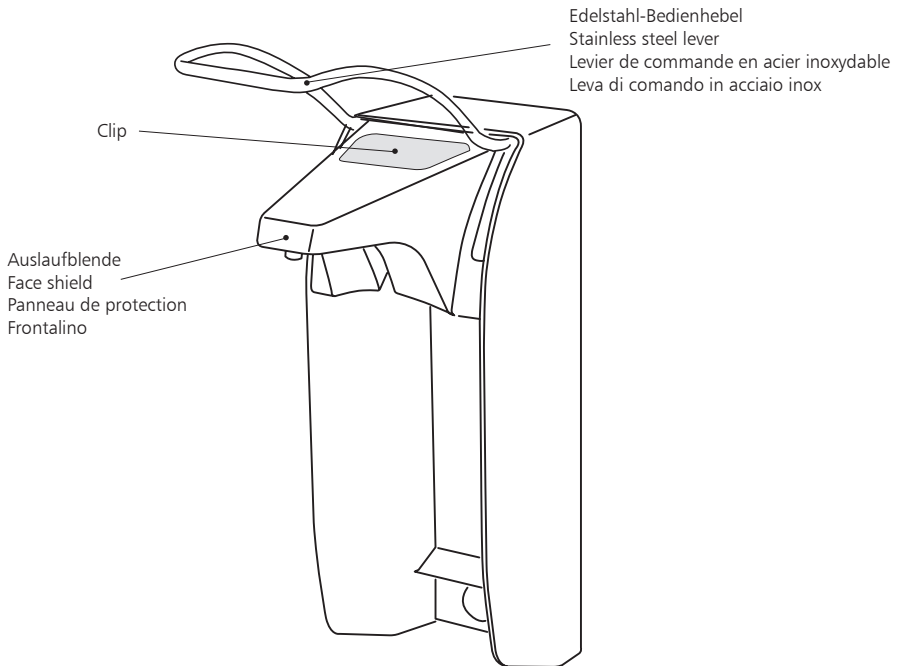
# MEDCARE

Seifen- und Desinfektionsmittelpender  
Dispenser for soap and disinfectant  
Distributeurs de savon et de désinfectant  
Dispenser di sapone e di disinfettante

3600003339	MEDC05L
3600003340	MEDC05S
3600003341	MEDC10L
3600003343	MEDC10S

**MONTAGEANLEITUNG UND SICHERHEITSHINWEISE**  
**INSTALLATION AND SAFETY INSTRUCTIONS**  
**INSTRUCTIONS DE MONTAGE ET CONSIGNES DE SÉCURITÉ**  
**ISTRUZIONI PER L'INSTALLAZIONE E LA SICUREZZA**

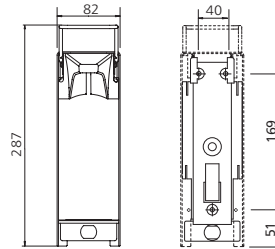
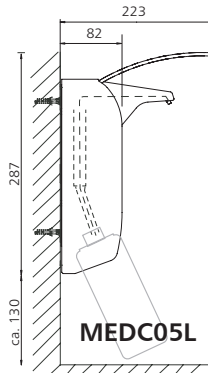
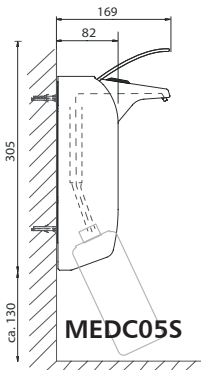
**KWC**

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### 500 ml

Wandhalter

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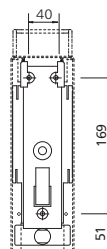
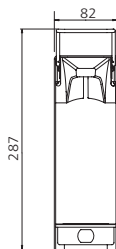
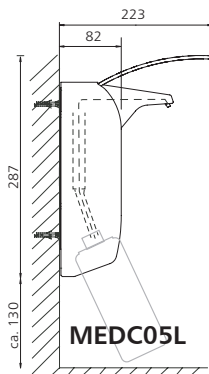
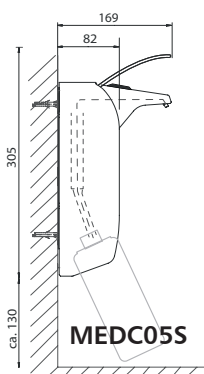
**Technical Drawing**

Technical modifications reserved.





**500 ml**

Wall plate

Dimensions in mm

**1000 ml**

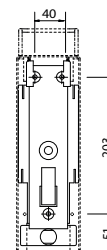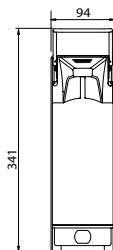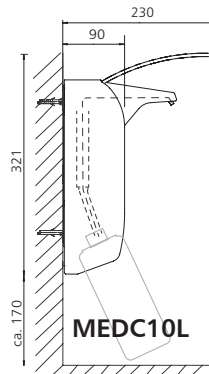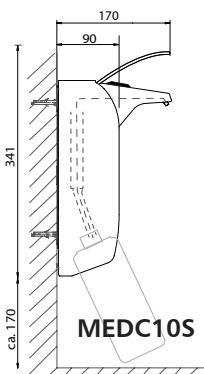Wall plate


Dimensions in mm

**Scope of Delivery**

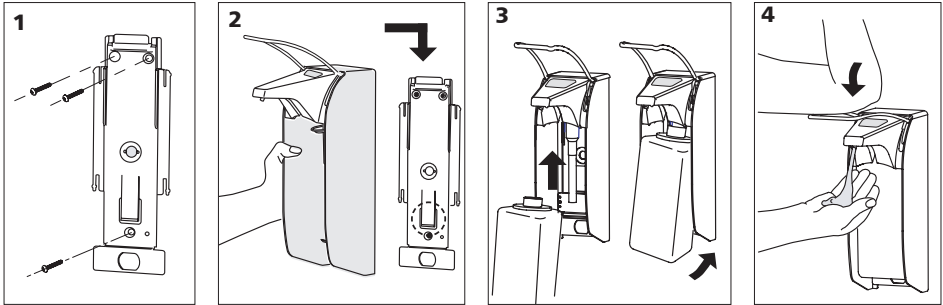Dispenser for soap and disinfectant without container and pump

Fixing set: screws and plugs

Instruction manual

### Important!

Please carefully read the operating instructions and save them for reference. The proper function of this dispenser system can only be guaranteed if the procedures in these instructions are followed.

The compatibility of the soap/disinfectant material used in this dispenser must be checked prior use, in order to guarantee fault free functionality. We assume no liability for any damages or costs which arise due to the incompatibility of filling material and dispenser systems.

## 1. Preparation

When using as a soap dispenser, install the dispenser as close as possible to the sink to prevent wet hands from dripping onto the floor.

The dispenser is mounted to a vertical surface using 4 screws. An optional 2 screws can be installed to permanently secure the dispenser to a vertical surface, if required.

Required space above the dispenser: 50-70 mm for the lever, and for dispenser removal and placement, as well as an additional 100 mm for operation. The required room below the dispenser, necessary for bottle removal, is about 300 mm.

Optimal mounting height for use: 130-135 cm, measured from the floor to the upper edge of the dispenser housing.

## 2. Mounting of the dispenser

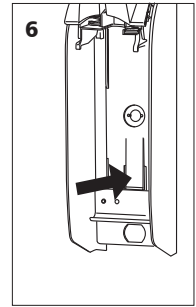
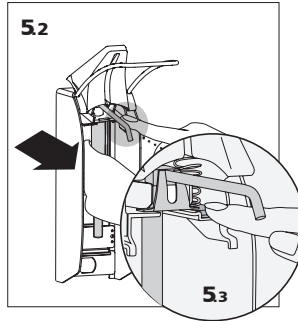
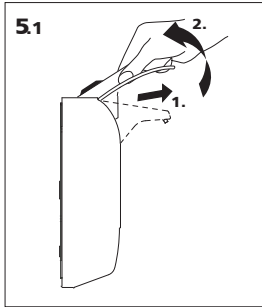
Remove the dispenser from the wall plate: press the latch on the back wall of the dispenser and slide the dispenser upwards, off the plate. With the enclosed, or other suitable attachment materials, attach the wall plate to the wall. Slide the dispenser onto the wall plate from the top until the latch clicks into place (see image 2).

## 3. Bottle Placement / Exchange

Insert the filled bottle into the dispenser from the bottom, until snaps into place.

## 4. Initial Operation

For first use hold your hand under the nozzle and operate the lever several times until solution is dispensed.

## 5. Pump Removal / Exchange

Gently pull face shield forward and then lift upwards. Next, press down on the pump lock with a finger, and pull the pump forwards (see images 5.1-5.3).

**⚠ Important!** Do not take apart the housing! Use only original replacement parts issued from the manufacturer. Only for the dosage of suitable solutions such as soaps, lotions and disinfectants. Caution notice! The use of unsuitable solutions can influence the function of the dispenser and may cause the solution to spray into eyes.

## 6. Cleaning / Sterilization

Remove the dispenser from the wall plate: to do this, press the latch on the back wall of the dispenser and slide the dispenser upwards, off the plate.

The MEDCARE dispenser and stainless-steel pumps (outlet pipe made from stainless steel), are autoclavable with superheated steam (134 °C, 2 bar, max. 20 min dwell time).

**⚠ Important!** Wall plate, bottle and plastic pumps (K pump) are not autoclavable! For pump removal, see step 7. After autoclaving dispenser and the stainless steel pump, allow it to cool for at least an hour prior to next use.

## 7. Information for General Care

The Robert Koch-Institute recommends: "For both dispenser types (note: hand disinfectant and soap dispenser), the fixed outer and inner parts must be able to be reprocessed. This is also possible by machine. [...] Visible contamination of the outer housing and all accessible parts without further manipulation as well as soilings at the outlet must be removed by wipe disinfection. Daily wipe disinfection is recommended for operating levers. The scope and frequency of checks and the reprocessing of dispensers should be specified in the internal hygiene plan, e.g. every six months." [Source: Robert Koch Institute, recommendations for hand hygiene, 2016]

In order to guarantee proper functioning of the dispenser regular manual cleaning or processing in cleaning-disinfection devices is necessary.

Manual cleaning: Regularly remove adherent product residual. Cleaning of the dispenser housing and the pump with hot water. Rinsing the pump with hot water. Disinfection of shroud is possible with appropriate surface disinfectants.

Processing in cleaning-disinfection devices: choose program „Dispenser cleaning“ and use pH-neutral cleaning solutions. Important: Cleaning of plastic pumps up to max. 65 °C!

Processing in autoklave: the dispenser and stainless steel pump are autoclavable with superheated steam (134 °C, 2 bar, max. 20 min dwell time).

Plastic wall plate, bottle, plastic pumps (K pumps) are not autoclavable.

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KWC ME LLC Ras Al Khaimah,
KWC DVS Ltd - Northern Office
KWC DVS Ltd - Southern Office
KWC Aquarotter GmbH
KWC Northern Europe Oy
**KWC Austria GmbH**